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GB Congratulations!

You have selected a magnifier from SCHWEIZER, a quality product manufactured according to the currently valid standards and regulations. A „Made in Germany“ brand. Congratulations on your decision to buy it.

Please read these instructions for use completely before using your magnifier for the first time and observe the safety and care instructions.

GB General notes

The lens in your SCHWEIZER magnifier is made of glass or acrylic glass. The optical quality of both materials is second to none. Lenses manufactured from optical glass are ground and optically polished on both sides. The hard surface features a high chemical and mechanical resistance. Only UV-absorbing crown glass is used in the manufacture - the same glass that is used for the manufacture of prescription spectacles.

Lightweight lenses (acrylic glass lenses) are manufactured from optically pure Plexiglas®. Computer calculated, aspherical surfaces ensure optimum edge-to-edge image quality. The entire field of vision is thus optimally utilised.

Caring for your magnifying glass

Proper care of your magnifier will allow it to provide you with many years of untroubled reading pleasure.

Clean the lens with a damp cloth (ideal: a damp micro-fibre cloth). When cleaning a lightweight lens (acrylic glass lens) never use solvents such as alcohol, benzene or other cleaning agents as these can damage the lens. Small lines that look like hairline cracks at the edges of lightweight lenses or in the plastic frame are flow lines that occur during processing of the material. They do not affect the function or durability in any way.

General safety notes

Fire hazard! Never leave your magnifier out in the sun uncovered. The intensifying effect of the lens can start a fire. Risk of injury!

- Never use the magnifier to look at the sun.
- Do not directly look at the light source with illuminated magnifiers.
- Avoid mechanical effects such as impacts, banging or dropping on the floor, as well as moisture and high temperatures.

Inform other people about these risks, especially children. Do not dispose of old batteries with the household waste. These need to be disposed of properly (see page 13).

Power supply

The mains-operated version of the illuminated magnifier TWIN-Lux LED requires mains voltage. The series ERGO-Lux i mobil contains a lithium polymer battery.

The ÖKOLUX plus mobil and the ERGO-Lux MP series require three standard alkaline batteries; Type Micro, 1.5 Volt (LR3, AAA, AM-4).

The ÖKOLUX plus, MODULAR LED and TWIN-Lux LED series requires three standard alkaline batteries; Type Mignon 1.5 Volt (LR6, AA, AM-3).

The LED light-emitting diodes are subject to the most stringent selection criteria. They have a lifespan of more than 50.000 hours with continuous use. A replacement of the LED is thus normally not required.

GB Batterie safety notes

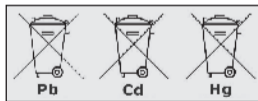
We would like to inform you that our batteries are marketed in accordance with the current law and refer you to the following points:

- To prevent harmful effects to the environment and health, used batteries must be disposed of properly at community recycling centres or returned to the merchant. Returning batteries is required by law and is completely free-of-charge for you.
- Batteries should only be returned in a discharged state.
- Valuable raw materials such as zinc, iron and nickel can be reclaimed by recycling used batteries.
- The batteries can be easily removed from our devices for disposal (see User's Guide). Magnifiers with

rechargeable batteries which cannot be removed please dispose completely.

- The symbol of a „crossed-out trash container“ indicates that it is illegal to dispose of used batteries with regular domestic waste.

The additional specification of the chemical sign for lead (Pb), cadmium (Cd) and mercury (Hg) indicates that these materials are contained in the batteries in accordance with the legally-specified limits.

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GB For the following products:

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ÖKOLUX plus mobil, SEMPRAL, FUNCTIONAL, ERGO-Lux MP mobil,
ERGO-Pocket

GB And here's what else you'll find at SCHWEIZER:

- Europe's largest range of special corrective lenses
 - Special spectacle lenses for increased contrast
 - Special spectacles with magnifying effect
 - Magnifying spectacles
 - Telescopic magnifying spectacles
 - LowVision illumination and much more
- Electronic Aids
- Magnifiers for technical applications
 - Precision fold-in magnifiers
 - Precision measuring magnifiers
 - Scale measuring magnifiers
 - Thread counters
 - Wand microscopes
 - Watch makers' magnifiers
 - Work stand magnifiers, illuminated

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Copyright © Januar 2017 A. SCHWEIZER GmbH
- Ⓒ All rights reserved.